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**Thank you for purchasing Ergonova**

Please read this manual carefully before using and keep it properly for future reference.

## Introduction

### Introduction, scope and groups of application

EyeLime eye massager with its human-based design 180 degrees folding body design, convenient and easy for carry and storage. It adopts intelligent air pressure, micro-computer chip control technology. By kneading and massaging on eyes , it helps to add oxygen, lead meridian, promote the blood circulation of eyes, ease the pressure and fatigue, and accelerate metabolism to keep your eyes in the healthiest state.

### Scope of application

It kneads and massages the specified acupuncture point, so as to ease the pressure and fatigue, Improving nearsightedness, releasing eyestrain, and preventing nearsightedness. It has the effect of short sight prevention and can relief tension and pressure.

### Acupuncture point and massage

#### groups of application

Those suffering from near-sighted, weak-sighted or astigmatic

Those suffering from sight fatigue, pseudomyopia

Those suffering from insomnia and dream and neurasthenia


Those using eye for long time and feeling sight fatigue

Those suffering from black rims or crow's-feet


## Product configuration

### Product configuration

The massager includes: master unit, manual controller, power charger, battery, user manual, earphone, certificate of approval and warranty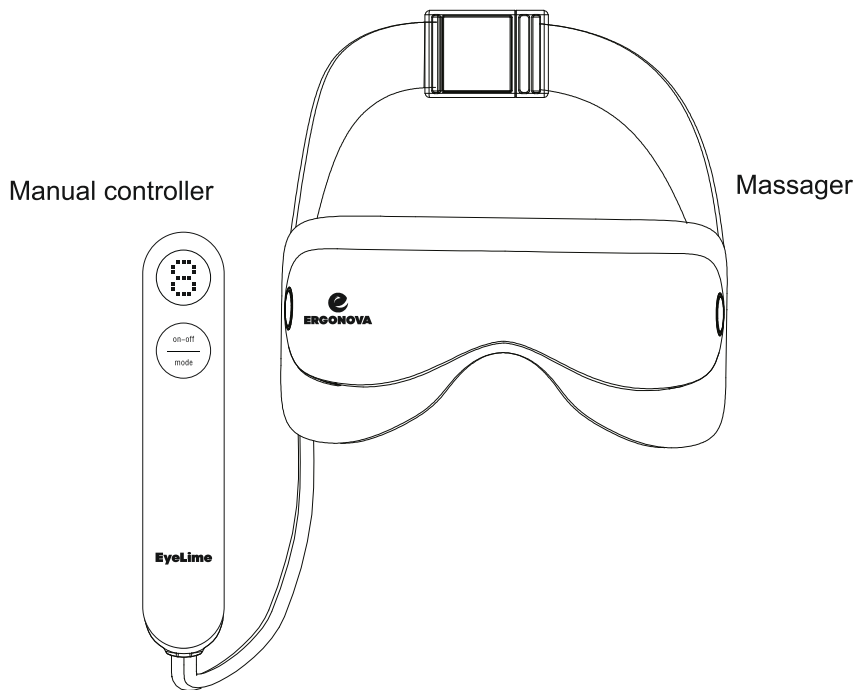

3 batteryes AA

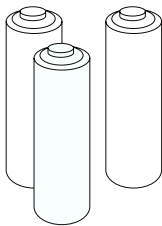
User manual

## Description and Features

1. Zone mode display of air pressure / heating
2. Power / select heating /power Off
3. Button to switch the air pressure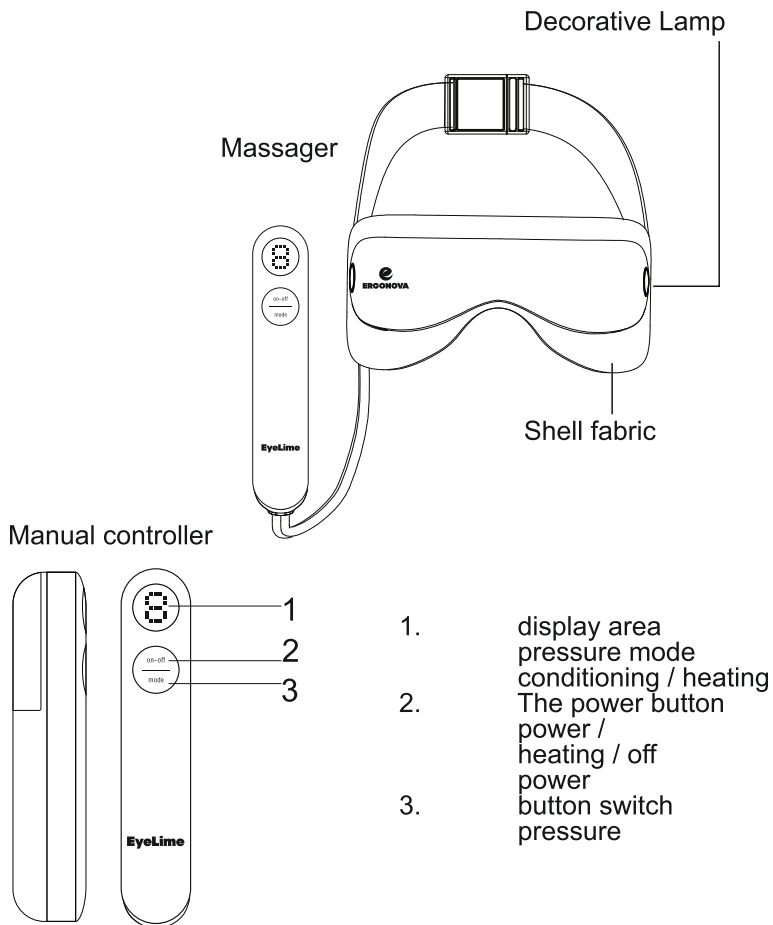

## Preparation & Operations

### Preparation

Please check the integrity of the machine and accessories.  
No damage on connection between handle and master unit  
Enough power  
No damage on power charger

### Operation

#### Power on

#### Installing the battery

Open the battery cover on the back of the manual controller  
Input 3 AA batteries in accordance with the anode and cathode indication  
Close the battery cover.

#### Power charger

Insert the column plug of the power charger into the power jack of the handle, while the other end is connected to the alternating current power supply.

#### Power adapter socket

#### Wear

Please at the eyes, wear the eye massager  
Adjust buckles and comfortable


## Operations

Operation


Default

air pressure 1 + low temperature foment massage for 15 minutes.  
Change the time, foment and air pressure mode by operating relative function keys manually according to demand.

Manual mode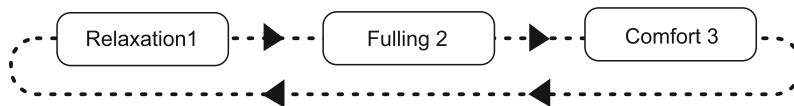

Fomentation – press –Low temperature/ temperature /high temperature

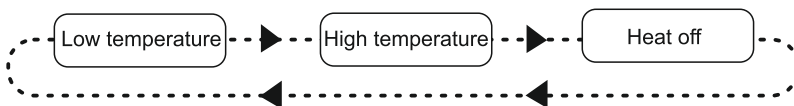## Precautions

Please do read this User's Manual carefully for correct and safe use of this massager.

1. Please do not use it when driving.
2. Please do not use this product in the environment with high temperature or humidity.
3. Be sure of taking off the contact lens before using.
4. Wear the device and then switch it on to avoid damaging the air bag due to overpressure.
5. Those suffering from eye diseases, such as glaucoma, pearl eye and detachment of retina can not use this product.
6. Those who have eye surgery can not use this product.
7. The persons suffering from strokes, hypertension, and hypotension do not use this product.
8. Sluggish patients, or people with physical or mental or intellectual disability are not allowed to use it.
9. Please use the DC4.5V stabilized voltage power supply provided by our company and pay attention to the safety. Battery can not be put into the battery box if using the external power supply.
10. Please take out the battery if its power is not enough or it will not be used for a long time, so as to avoid damage to the machine due to leakage of the battery fluid. Please put the waste battery into the designated recycling drum to avoid environment pollution. (Alkali battery is recommended.)
11. It is household electric health care product, thus can not be used for diagnose and cure or other commercial use.
12. If it requires cleaning, wipe it with cotton soaked in alcohol. Do not use water to wash the device.
13. When folding the main unit, never put anything in the center, or touch the center with your finger or any part of the body, to avoid damage or injury.

## Troubleshooting

### Troubleshooting

If there is any problem when you use the device, please follow the instructions below to check. If the problem still exists or the device can't work normally, please turn off the power supply and contact local service center. Do not disassemble or repair the device yourself.

Fault	Possible reason	Solutions
Unable to boot	Whether the cable is in Good condition	Check the status cable
	Whether battery is low	If yes, change battery
	Whether the battery is correctly installed	Please ensure the right electrode; make sure the battery is installed and fixed in the battery holder
	Whether the power charger is connected properly	Please ensure it is connected properly
Air press fomenting is too weak	Whether the battery is low	If yes, change battery <input type="checkbox"/> Make sure the battery is installed and fixed in the battery holder

Technical parameters ∠Power ∠Ideal environment ∠Dimension ∠N.W. ∠Power ∠Specifications	DC: 4.5V 1000mA Temperature: 5°C-40°C Relative Humidity: ≤80% Master unit: Length220mm×Width100mm×Height86mm Manual controller : Length 184 mm×Width 36mm×Height 39mm 458g 4.5W Magnetic field strength:50mT~250mT Fomentation:30°C~45°C
--	--

## Storage

### Storage

In case the storage period exceeds 6 months, please check it and power it on for half an hour.

Do not store close to floor, wall or roof.

Keep good ventilation condition and away from direct sunshine and corrosive gases.

### Transportation

The product has simple shockproof design and can be transported by air, railway, highway or ship; avoid rain, snow, impact, and always keep the right side up.

### Checking the power supply

-Check the power supply and do not switch on the device if the voltage exceeds the rated range of the device.

-Check the eyes cover and cable regularly. Stop using the device immediately if there is any breaking or rupture. Please send it to local service center.

### Maintenance

Take measures to prevent moisture, rats and pollution.

Switch off the device before cleaning the manual controller.

Put the device in safe place after using. Do not put heavy objects on the device.

Contact qualified technician if there is any fault. Do not disassemble the device yourself.

The warranty does not include batteries.

The warranty does not cover the following conditions.

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Авторское право © 2012г.